

Biolog 3000i seamlessly integrates with EMIS LV


Paperless storage and the transfer of ECG data from the Biolog 3000i to the EMIS LV system is now even easier with the launch of Cardioview Integration.

Biolog 3000i is a popular handheld ECG machine that is bringing the paperless practice one step closer to general practice.

The popularity of the Biolog 3000i is due to its compact size, simple operation, easy to use Cardioview software and the ability to file electronic copies of the ECG files directly into patients' notes.

Using Cardioview Integration, the Biolog ECG process has now been fully automated and is controlled entirely from the patient medical record within EMIS LV.

Full EMIS LV integration means that practices can now file ECG results directly into the patient medical record. Until now the process of attaching the ECG comprised several steps, including manual ECG transfer from the Biolog to the Cardioview software, and manual attachment into the medical record.

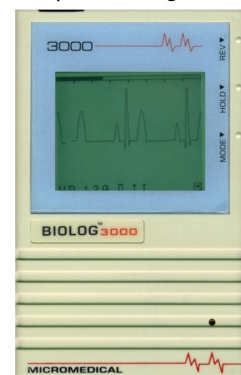

Biolog 3000i handheld ECG machine

Simple to use

Transferring ECG data from the Biolog 3000i direct to an EMIS LV patient medical record is simple.

- Simply click the shortcut icon that is displayed on the EMIS LV toolbar, confirm the patient details are correct, and the ECG will be automatically transferred from the Biolog 3000i
- Following successful completion of the ECG transfer, Cardioview software will display the ECG on screen, enabling the user to check the quality
- One further mouse click and the ECG will be automatically filed and attached against the patient medical record.



ECG displayed using Cardioview software

Once attached within the EMIS LV medical record, the ECG can be viewed on any Windows based LV terminal in the practice, as with any normal scanned document.

Cardioview Integration is available for both new and existing users.

Further information

If you would like more information on the Cardioview Integration or the Biolog 3000i, please visit www.numed.co.uk alternatively telephone Numed Cardiac Diagnostics on **0114 243 3896** or email info@numed.co.uk

NUMED

CARDIAC DIAGNOSTICS