



CardioSoft 12 Lead ECG System

Easy User Guide

support.numed.co.uk

1

Connect the CardioSoft ECG System to your PC or Laptop.



2


Open the patient's clinical record in your clinical system.



Note: INPSHosted and AEROS sites must ensure that Numed Enterprise Link software is running on the remote desktop


3

Double click the I3 icon, this is usually found on your taskbar or desktop. This will launch the I3 software.

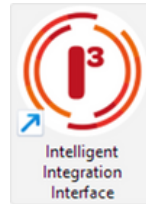


4

Click the 'Start' button.

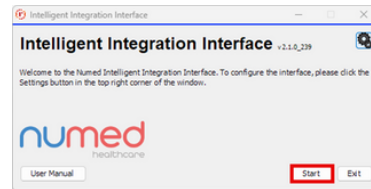


5

Select 'ECG' from the product selection screen - if you have more than one ECG system you may need to select 'GE CAM-USB'.

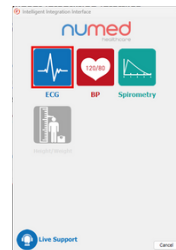


6

Confirm the patient's details are correct then click 'Yes'.

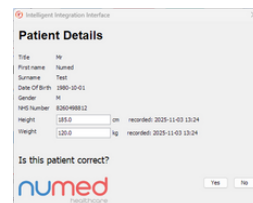


7

The patient's ECG will then appear on screen.

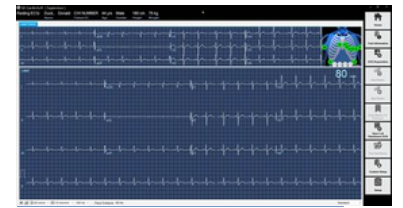

8

Once you are happy with the ECG trace, press the 'ECG Acquisition' button. There may be a delay of up to 10 seconds whilst the ECG is acquired.

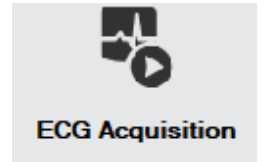

9

Once the acquisition process has completed press 'Post Test Review'.




10

To begin the filing process, close the CardioSoft software by clicking the red cross in the top right hand corner.




11

Add any comments and then click 'Send'.

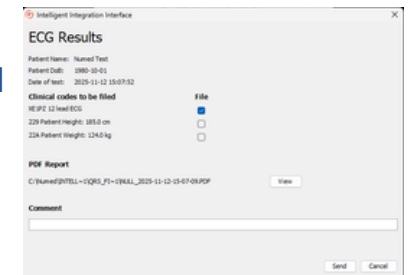

12

Click 'OK' to complete the process.

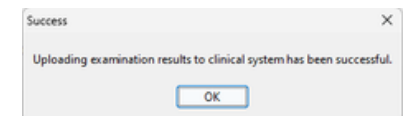

13

The ECG will now be filed into the patient record in the clinical system.

**Keeping your service
running smoothly**

numed | Dedication
healthcare | Innovation
Trust