



MAC 600 ECG
Machine without
Integration

support.numed.co.uk

Easy User Guide

To use your GE MAC 600 ECG machine and print out the results without integrating a PDF back to the clinical system, please follow the steps below.

Normal 12-lead ECG report

1

Connect the patient leads to the MAC 600 ECG system ensuring that they are correctly oriented and fully inserted.



2

Power on the device.




3

Press the 'F1' key to access the patient details screen.



4

Enter your patient's details using the keypad and arrow pad. To confirm the details, press the white circle which is located in the centre of the arrow pad.

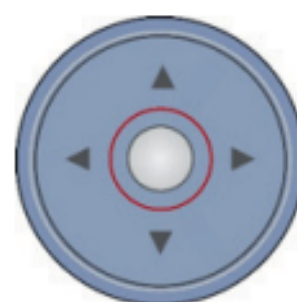

* We recommend that NHS customers use the NHS number or CHI number if in Scotland.

5

To check the quality of your ECG, use the trace indicator, which is located in the top right of the screen.



* Green symbolises good connectivity with the patient. Orange symbolises poor quality due to interference or poor connection to the patient, and Red indicates possible lead disconnections.

6

To capture your ECG trace, press the green 'ECG acquisition' button.



7

If the captured ECG meets your requirements, press the 'F1' button to print the report out.



* If you would like to store your ECG on the machine, press the 'F3' button before printing.

8

Power off the system and remove the lead wires from the patient.



Printing a 3-lead rhythm strip

To print out a 3-lead rhythm strip, first hook up your patient as normal, then press -



This is below the green button.

The ECG will keep printing until you press -



This is below the white button.

**Keeping your service
running smoothly**

numed | Dedication
healthcare | Innovation
Trust