



SpiroConnect
Cleaning Guide

support.numed.co.uk

Body cleaning instructions

After each use the unit should be wiped down with a damp and not overly moist alcohol infused wipe or other medically approved anti-viral wipes.

Take care to ensure that no liquid enters the unit itself.

Turbine cleaning instructions



The frequency of turbine cleaning and sterilisation is up to the users' own infection control policies and procedures.

Remove the turbine by rotating the mouthpiece holder assembly anti-clockwise by 90o and gently pulling from the main body. Take care not to over-twist the green dome as this may cause damage.



Routine cleaning

For routine cleaning, the turbine can be submerged in warm, soapy water, rinsed, and then left to air dry (see drying guidance below).

Disinfecting the Turbine

The turbine can be disinfected if required using RelyOn+ PeraSafe. The solution must be made up as per the manufacturer's instructions (see below). The turbine can then be soaked for 10 minutes only - do not exceed this time - and rinsed using drinking water*. It is vital that you rinse off the PeraSafe solution. Once rinsed, the turbine can be left to air dry (see drying guidance below).

*If limescale deposits form on any part of the turbine, then de-ionised, distilled or sterile water should be used instead of drinking water.



Please note that no other sterilising solution should be used, as wrong or improper use can cause damage to your turbine which isn't covered under warranty.

Drying Guidance

The turbine must only be left to air dry; please do not use cloth, paper towel or cotton buds to dry your turbine. Do not wrap your turbine in paper or cloth either. Do not stick anything inside the turbine tube at the mouthpiece or bottom ends.

RelyOn+ PeraSafe mixing instructions

Mix 1 sachet or 16.2g PeraSafe with 1 litre of lukewarm (30-35°C) water until all powder has dissolved and the solution has turned a clear blue/green colour, PeraSafe is fully activated after 15 minutes, and the solution will go clear after approximately 30 minutes.

The solution can be used once activated and should be stored at room temperature. It is recommended that PeraSafe is prepared not more than 1 hour before use. The solution should be discarded after 24 hours or 8 hours after 1st use, whichever comes first.

**Keeping your service
running smoothly**

numed | Dedication
healthcare Innovation
Trust