

# SpiroConnect Cleaning Guide

## Body cleaning instructions

After each use the unit should be wiped down with a damp and not overly moist alcohol infused wipe or other medically approved anti-viral wipes such as Clinell wipes. Take care to ensure that no liquid enters the unit itself.

## Turbine cleaning instructions

The frequency of turbine cleaning and sterilisation is up to the users own infection control policies and procedures.

Remove the turbine by rotating the mouthpiece holder assembly anti clockwise by 90° and gently pulling from the main body. Take care not to over twist the green dome as this may cause damage.



DEDICATION • INNOVATION • TRUST

Alliance House, Roman Ridge Road, Sheffield, S9 1GB Telephone: 0114 243 3896 Fax: 0114 243 3413



[www.numed.co.uk](http://www.numed.co.uk)



[info@numed.co.uk](mailto:info@numed.co.uk)

Numed Holdings Limited T/A Numed Healthcare, Company Reg No 01302868

Directors: M K Healy & S J Healy

### **Routine cleaning**

The turbine can be submerged in warm soapy water for routine cleaning, rinsed and then left to air dry (see drying guidance below).

### **Sterilising the Turbine with RelyOn+ PeraSafe**

To sterilise the turbine with RelyOn+ PeraSafe you must create the solution as per manufacturer instructions (see below). The turbine can then be soaked for 10 minutes only, do not exceed this time, then rinsed using distilled or sterile water. It is vital that you rinse off the PeraSafe. Once rinsed the turbine can be left to air dry (see drying guidance below).

**Please note that no other sterilising solution should be used, and wrong or improper use can cause damage to your turbine which isn't covered under warranty.**

### **Drying Guidance**

The turbine must only be left to air dry, please do not use cloth, paper towel or cotton buds to dry your turbine. Do not wrap your turbine in paper or cloth either. Do not stick anything inside the turbine tube at either mouthpiece or bottom ends.

### **RelyOn+ PeraSafe mixing instructions**

Mix 1 sachet or 16.2g PeraSafe with 1 litre of lukewarm (30-35°C) water until all powder has dissolved and the solution has turned a clear blue / green colour, PeraSafe is fully activated after 15 minutes, and the solution will go clear after approximately 30 minutes.

The solution can be used once activated and should be stored at room temperature. It is recommended that PeraSafe is prepared not more than 1 hour before use. The solution should be discarded after 24 hours or 8 hours after 1<sup>st</sup> use, whichever comes first.

DEDICATION • INNOVATION • TRUST

Alliance House, Roman Ridge Road, Sheffield, S9 1GB Telephone: 0114 243 3896 Fax: 0114 243 3413



[www.numed.co.uk](http://www.numed.co.uk)



[info@numed.co.uk](mailto:info@numed.co.uk)