

Easy User Guide

UNIVERSAL SMART ECG SYSTEM

1

Connect the **Universal Smart ECG System** to your **PC** or **Laptop**.

2

Open the patient's **clinical record** in your clinical system.



Note: INPS Hosted and AEROS sites must ensure that Numed Enterprise Link software is running on the remote desktop


3

Double click the I³ icon, this is usually found on your taskbar or desktop. This will **launch** the I³ software.



4

Click the **'Start'** button.

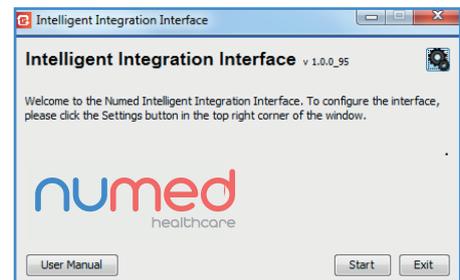


5

Select **'ECG'** from the product selection screen - if you have more than one ECG system you may need to select **'Universal ECG'**.




6

Confirm the patient's details are correct then click **'Yes'**.

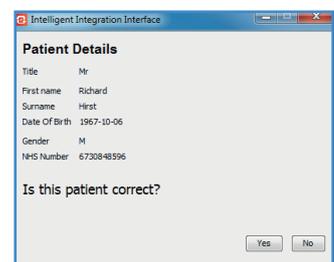


7

The patient's **ECG** will then appear on screen.

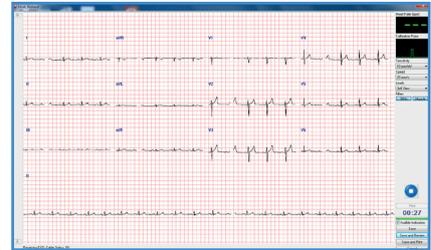

8

Once you are happy with the ECG trace, press the '**Save and Review**' button to transfer the most recent 10 seconds of ECG to **Cardioview**.

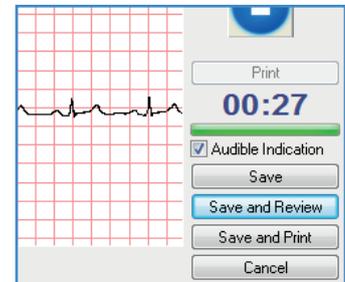


9

To begin the filing process, **close** the **CardioView** software by clicking the **red cross** in the top right hand corner.




10

A **comments box** will appear. Enter any additional information you would like recorded alongside the **ECG report** and click '**OK**'.

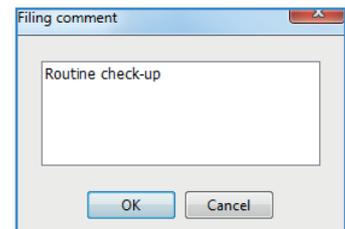

11

A **confirmation message** will be displayed when the **ECG** has been successfully uploaded to the patient record - click '**OK**' to complete the process.

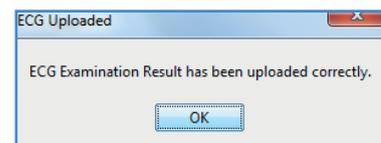

If you experience any problems using the **Universal Smart ECG System**, please contact our technical support team on: 0114 399 0010 or visit: www.numed.co.uk/support

numed | Dedication
healthcare | Innovation
Trust

Alliance House, Roman Ridge Road, Sheffield S9 1GB
Tel: 0114 243 3896 Fax: 0114 243 3413 info@numed.co.uk
www.numed.co.uk