




Universal Smart ECG System

Using Your Smart Universal ECG machine
without Integration

support.numed.co.uk

If your clinical system is down or you are not connected to the internet, follow the steps in the guide below to take, print, and export your ECG as a PDF.


 Before starting, please ensure that your ECG machine is plugged into the PC or laptop.



Taking your ECG


1

Launch CardioView from the desktop.

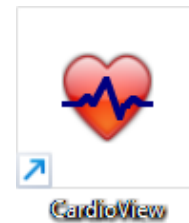


2

Click New Test.

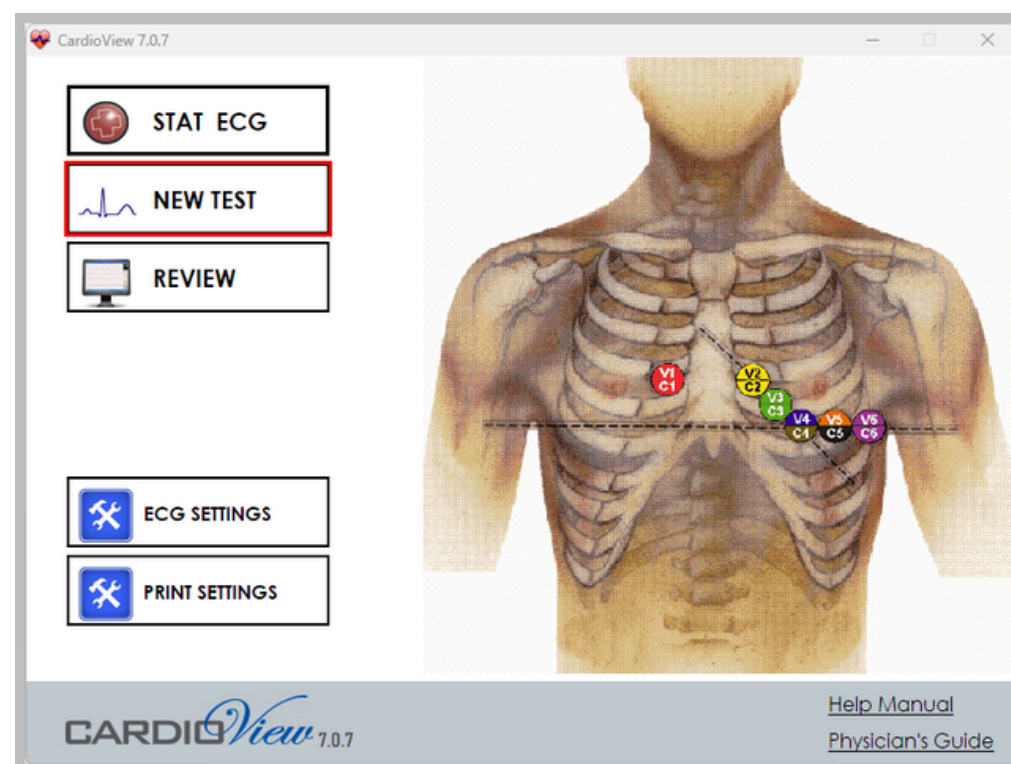

3

Fill the Patient Information form and click OK.

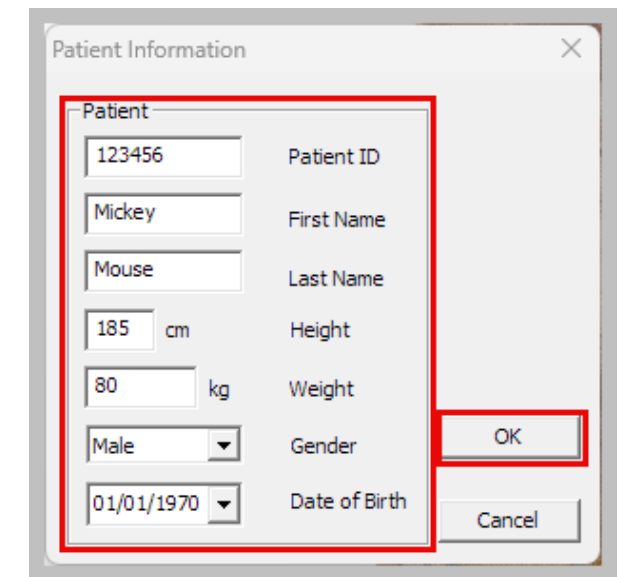


Patient Information form with fields for Patient ID (123456), First Name (Mickey), Last Name (Mouse), Height (185 cm), Weight (80 kg), Gender (Male), and Date of Birth (01/01/1970). The OK button is highlighted with a red box.



Please be aware that all fields are mandatory, including Height and Weight.


4

Click Save.

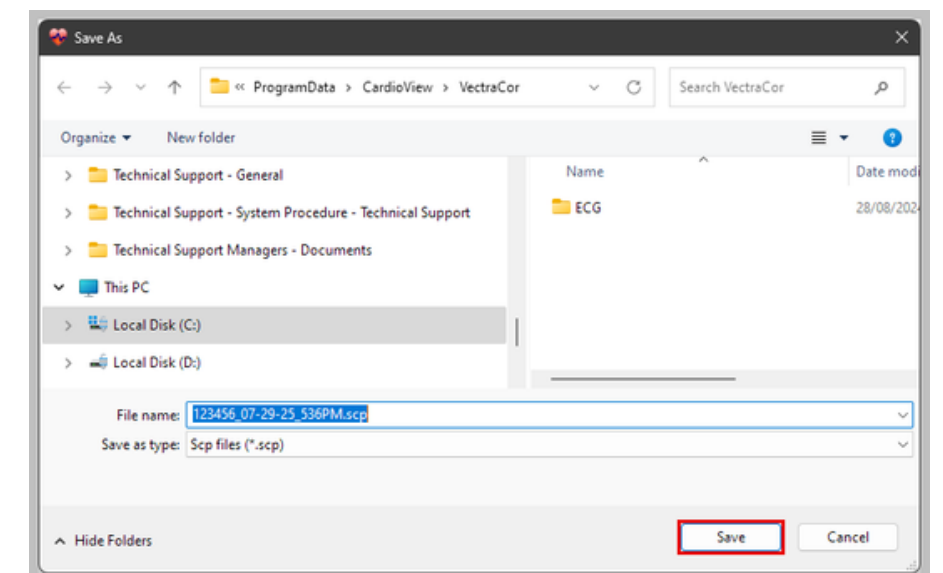

5

Connect the patient to the electrodes and start a trace on the screen. When satisfied, click Save and Review.

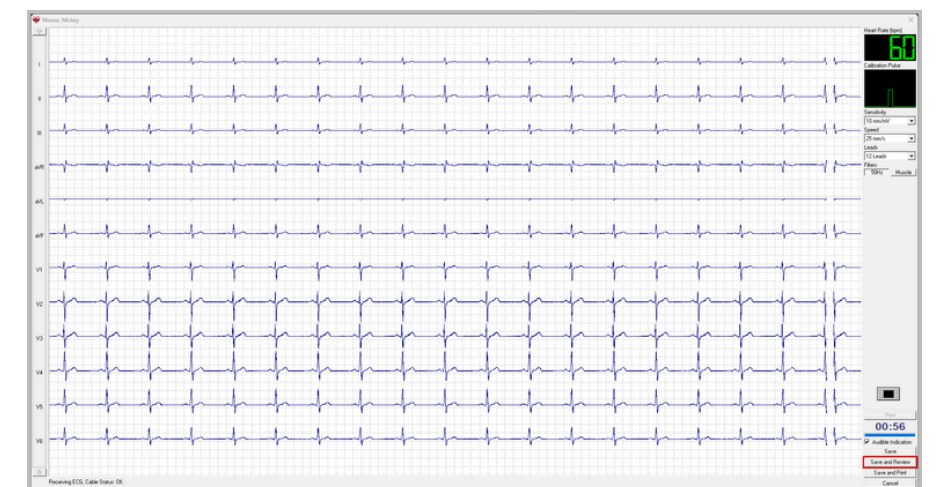


6

To print a paper copy of the ECG, click the print symbol in the top left corner.



7

To export the ECG report as a PDF file, click on the File menu and then select Print to File...

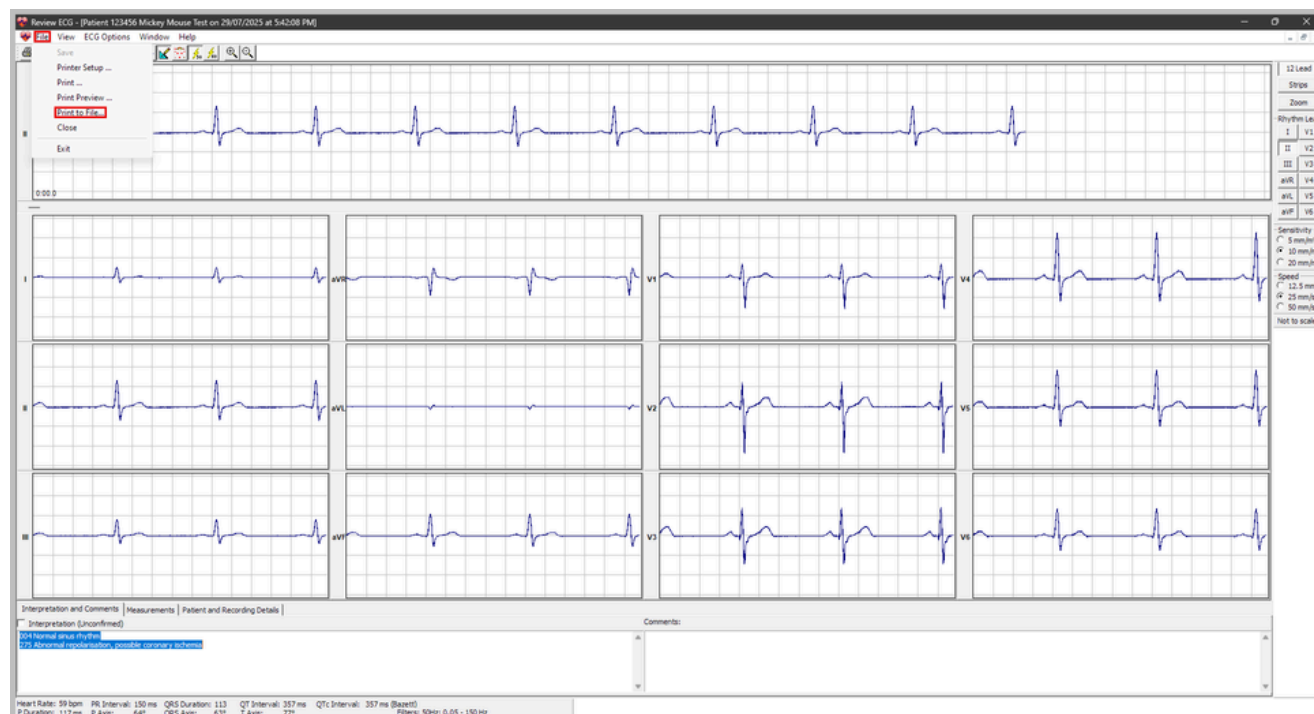

8

Click OK on Report Options.

Report Options

12 Lead Reports

☒ 3x4 Simultaneous

☐ 3x4 Sequential

☐ Averaged Complexes

☐ 6x2 Format

☐ 6x1 Format (2 Pages)

☐ Measurements Table

6 Lead Reports

☒ 3x2 Simultaneous

☐ 3x2 Sequential

☐ 6x1 Format (1 Page)

Single Lead Reports

☐ Strip

☐ Averaged Complex

Lead II

Scale 1x

Biolog Reports:

☐ Single Lead Biolog

Options

Speed 25 mm/s

Sensitivity 10 mm/mV

Minor Grid Dotted

Include

☒ Measurements

☒ Interpretation

☒ Comments

Report title:

Printer Setup...

Restore Defaults

Ok Cancel

9

Choose the location where you want to save the PDF, enter the file name, and click Save.

Save As

Save in: ECG Results

Name

Date modified

Type

No items match your search.

Home

Desktop

Libraries

This PC

Network

File name: Mickey Mouse ECG

Save as type: PDF Files (*.pdf)

Save

Cancel

☒ View PDF File

10

Once you regain access to the clinical system, you can attach the PDF to the patient's record.

**Keeping your service
running smoothly**

numed | Dedication
healthcare | Innovation
Trust